



Microbiology

ELI COLOREX™ VET ID



BACTERIAL IDENTIFICATION AND ANTIBIOTIC TREATMENT GUIDANCE

Tri-compartmented agar for the direct detection and identification of bacteria and their antibiotic susceptibility in :

- *mastitis and diarrhoea infections **in calves.***
- *ear, urinary, skin and digestive tract infections **in cats and dogs.***

- ◆ Quick and **easy-to-use** in 2 minutes.
- ◆ Validated on **several samples**: urine, stool, wound, milk.
- ◆ **Visual interpretation** and results within 24 hours.
- ◆ Compartment with AMC antibiotic for sensitivity testing.

ELI COLOREX™ VET ID


► PRINCIPLE

The Gram + and Gram - compartments allow the isolation and differentiation of bacteria. They contain respectively an inhibitor of Gram + and Gram - bacteria, and a mixture of chromogenic substrates whose hydrolysis allows direct identification of the main species by spontaneous coloration of the colonies within 24 hours at 37 °C ± 1 °C.

The "AMC" compartment is a non-selective agar allowing the isolation and differentiation of Gram + and Gram - bacteria resistant to Amoxicillin + Clavulanic Acid (AMC).

► PROCEDURE

- 1 **Plate** the 3 compartments of the agar **ending with the AMC compartment** according to the type of sample.
- 2 **Incubation** 24h at 37 °C ± 1 °C.
- 3 **Read the results** according to the chart provided.

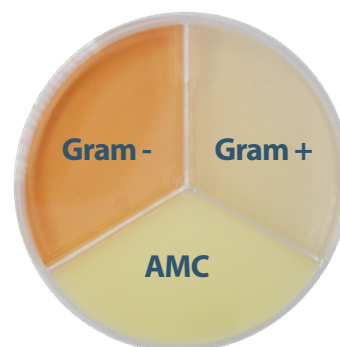

► RESULTS INTERPRETATION

Some examples

Gram (+)	Gram (-)	Compartment AMC
<i>Enterococcus spp.</i>	<i>Escherichia coli</i>	<i>Escherichia coli</i>
<i>Listeria spp.</i>	<i>Citrobacter freundii</i>	<i>Klebsiella / Enterobacter</i>
<i>Staphylococcus aureus</i>	<i>Enterobacter</i>	<i>Serratia</i>
<i>Staphylococcus spp.</i>	<i>Klebsiella pneumoniae</i>	<i>Citrobacter</i>
<i>Streptococcus uberis</i>	<i>Serratia marcescens</i>	<i>Proteus mirabilis</i>
Other species	<i>Pseudomonas spp.</i>	<i>Pseudomonas aeruginosa</i>
	Other species	<i>Staphylococcus aureus</i>
		<i>Streptococcus spp.</i>
		<i>Enterococcus</i>

► PRODUCT RANGE

REF.	NAME	QTY
22803	ELI COLOREX VET ID	3 Tests



WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country.



: ELITech Microbio (France)

Headquarters
Australia
BENELUX

+33 1 41 45 07 10
+1800 815 098
+31 313 430 500

France
New Zealand
United States

+33 4 83 36 10 82
+0800 555 611
+1 435 0752 6011



ELITechGroup

EMPOWERING IVD

www.elitechgroup.com
em-marketing@elitechgroup.com